





# Central Region N95 Selection

**Note:** This information is subject to change based on inventory levels.

**Background:** N95 mask selection should ensure appropriate seal for each individual while conserving supply. To ensure availability of appropriate masks for all, please follow the below guide to select an appropriate N95 Mask.

DESCRIPTION	IMAGE	INTENDED USER
<b>Manufacturer:</b> 3M <b>Model:</b> 8110S & 8210 <b>Size:</b> Small, Regular		All staff/providers who are able to achieve a proper seal <sup>1</sup> .  <b>Note:</b> Both of these respirators must be worn with a face shield for splash protection.
<b>Manufacturer:</b> 3M <b>Model:</b> 1860S & 1860 <b>Size:</b> Small, Regular		All staff/providers who <b>cannot</b> achieve a proper seal <sup>1</sup> with the 8110S or 8210
<b>Manufacturer:</b> 3M <b>Model:</b> 1870 <b>Size:</b> One size		All staff/providers who <b>cannot</b> achieve a proper seal <sup>1</sup> with the 8110S, 8210, 1860S or 1860  <i>* These respirators are currently on backorder with manufacturer with a very limited supply on hand. Manufacturer has no current release date available and will be phased out.</i>
<b>Powered Air-Purifying Respirator (PAPR)</b>		All providers/staff who cannot achieve a proper seal with either of the above masks  <b>Note:</b> Reach out to your manager for directions on how to obtain a PAPR.

1. Refer to [Seal Check Instructions](https://nmi.nmh.org/wcs/blob/1390904945317/20200406-n95-respirator-seal-check-v3.pdf) on NMI to ensure proper fit.
2. If individuals have respirator fit concerns or cannot properly achieve a respirator fit, contact Corporate Health and make an appointment for a respirator fit test.