


Donning ClearMask Used to Facilitate Patient Communication

1. Perform hand hygiene, remove your universal mask and store it in a clean, dry paper bag.
2. Perform hand hygiene and remove a single ClearMask™ from packaging (peel off the protective layer).
3. Position the mask with the foam facing towards you and the nose cutout on top (*see picture*).
4. As you place the mask on face, ensure your nose fits into the nose cutout and your chin rests on the bottom foam piece.
5. With your other hand, take both straps and place them over your head.
 - Place the bottom strap at the base of the head, below your ears.
 - Place the top strap higher at the back of the head, above your ears.
 - Do not crisscross straps.
 - To adjust, use the EZ-Adjuster (*see picture*) by pushing inward to tighten or pulling outward to loosen straps.

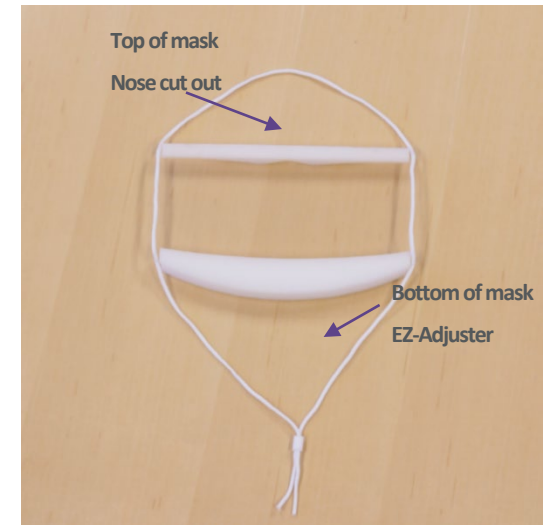

Removing ClearMask

1. Perform hand hygiene.
2. Remove the ClearMask™ by pulling the straps up and over the head. Loosen the adjuster piece if needed.
3. Slowly pull the mask away from your face. Avoid touching the front of the mask.
4. Store ClearMask in a clean, dry paper bag, and perform hand hygiene.
5. Don your universal mask.
6. Perform hand hygiene.

